Cavitron® Systems, Sterilizable Handpieces and Reservoir Bottles Infection Control Procedures

The objective of the information provided in this booklet is to supplement published general guidelines for reducing the potential for cross contamination when using a Cavitron® Ultrasonic Scaling and/ or Air Polishing Systems, Dispensing System, Cavitron sterilizable handpieces or reservoir bottles during routine dental care. In the event any regulatory agency disagrees with this information, the agency requirements take precedence.

For Infection Control Procedures relating to Cavitron ultrasonic inserts or air-polishing nozzles and inserts, refer to the procedures provided with each instrument.

Cavitron System Infection Control Procedures	
Warnings	<ul style="list-style-type: none"> Cavitron system cabinets, power cords, non-sterilizable handpieces, handpiece cables, foot controls, foot control cables, and Sterile Lavage Kit infuser bag are not sterilizable but can be disinfected following the procedures and recommendations listed below. Use water based non-immersion type disinfectant solutions. Alcohol based disinfectant solutions may be harmful and may discolor plastic material.
Limitations on Processing	<ul style="list-style-type: none"> Do not spray disinfectant solutions directly on system surfaces. Do not use disinfectant solution on sterilizable handpieces. Refer to sterilization procedure for the sterilizable handpieces.
Point of Use	<ul style="list-style-type: none"> Remove excess soil with disposable cloth or paper. Discard cloth after use.
Containment and Transportation	<ul style="list-style-type: none"> Handle with care.
Manual Cleaning	<ul style="list-style-type: none"> Generously spray disinfectant solution on a clean cloth. Wipe the outer surfaces of the cabinet, power cord, non-sterilizable handpiece, handpiece cable, air and water line, foot control, foot control cable and infuser bag. Discard used cloth. Wipe dry with a clean cloth.
Disinfection	<ul style="list-style-type: none"> Generously spray disinfectant solution on a clean cloth. Wipe the outer surfaces of the cabinet, power cord, non-sterilizable handpiece, handpiece cable, air and water line, foot control, foot control cable and infuser bag. Discard used cloth. Allow disinfectant to air dry.
Drying	<ul style="list-style-type: none"> When cleaning, wipe surfaces dry with a clean cloth. To achieve disinfection, allow surfaces to air dry.
Maintenance, Inspection and Testing	<ul style="list-style-type: none"> Visually inspect to ensure all contamination is removed. Visually inspect all cables and power cord for tears. Visually inspect handpiece cable o-rings for wear. Replace worn o-rings (refer to applicable Directions for Use of system). Lubricate o-rings with water before use.
Storage	<ul style="list-style-type: none"> Do not place on or next to a radiator or other heat source. System is portable but must be handled with care.

Steri-Mate 360 Handpiece Infection Control Procedures										
Warnings	<ul style="list-style-type: none"> All devices must be cleaned, disinfected & sterilized prior to each use. This also applies to the first use of devices supplied as non-sterile, as well as cases in which the sterile packaging has been damaged or opened. High level disinfection by itself is not appropriate for this device. Only use validated methods for cleaning / disinfecting and sterilizing. Only use cleaning and disinfecting solutions tested for compatibility with the device being disinfected. Tested disinfectants are listed showing active ingredient in parentheses. <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 33%;">Recommended Disinfectants:</td> <td style="width: 33%;"></td> <td style="width: 33%;"></td> </tr> <tr> <td> <ul style="list-style-type: none"> Isopropyl Alcohol 70% solution in water Bleach 0.5%-0.6% solution in water (0.5% to 0.6% sodium hypochlorite) Bi-Arrest (9.5% Ortho-Phenylphenol, 9.5% Ortho-Benzyl-para-chlorophenol) </td> <td> <ul style="list-style-type: none"> Cidex OPA (55% Ortho-Phthalaldehyde) Cavicide Synergized Superquat (84.5% Isopropanol, 15.3% Ammonium Chloride) Birex SE Phenolic with detergent (7.7% O-Phenylphenol, 7.6% p-tertiary amylphenol) </td> <td> <ul style="list-style-type: none"> Discide (63.25% Isopropyl Alcohol) Volo (41.6% Isopropyl Alcohol) Lysol IC (0.1% Alkyl Dimethyl Benzyl Ammonium Saccharinate, 79% Ethanol) </td> </tr> </table> <ul style="list-style-type: none"> Always use a pH neutral cleaning solution (such as Resurge® Instrument Cleaning Solution) / disinfecting detergent or solution. Product can be damaged by alkaline and acidic detergents. Read cleaning detergent / disinfecting solution instructions prior to use. Do not leave product standing in detergent or solutions. An automated method (washer / disinfecter) to clean and disinfect the device is recommended. Ensure equipment (washer / disinfecter and autoclave) is regularly serviced, inspected, and the validated parameters are maintained with each cycle. Do not allow device to exceed 279°F/137°C. Always observe all applicable legal and hygiene regulations for practice and/or hospital. Always wear protective gloves, glasses and a mask when handling contaminated instruments. Remove all packaging prior to processing. 	Recommended Disinfectants:			<ul style="list-style-type: none"> Isopropyl Alcohol 70% solution in water Bleach 0.5%-0.6% solution in water (0.5% to 0.6% sodium hypochlorite) Bi-Arrest (9.5% Ortho-Phenylphenol, 9.5% Ortho-Benzyl-para-chlorophenol) 	<ul style="list-style-type: none"> Cidex OPA (55% Ortho-Phthalaldehyde) Cavicide Synergized Superquat (84.5% Isopropanol, 15.3% Ammonium Chloride) Birex SE Phenolic with detergent (7.7% O-Phenylphenol, 7.6% p-tertiary amylphenol) 	<ul style="list-style-type: none"> Discide (63.25% Isopropyl Alcohol) Volo (41.6% Isopropyl Alcohol) Lysol IC (0.1% Alkyl Dimethyl Benzyl Ammonium Saccharinate, 79% Ethanol) 			
Recommended Disinfectants:										
<ul style="list-style-type: none"> Isopropyl Alcohol 70% solution in water Bleach 0.5%-0.6% solution in water (0.5% to 0.6% sodium hypochlorite) Bi-Arrest (9.5% Ortho-Phenylphenol, 9.5% Ortho-Benzyl-para-chlorophenol) 	<ul style="list-style-type: none"> Cidex OPA (55% Ortho-Phthalaldehyde) Cavicide Synergized Superquat (84.5% Isopropanol, 15.3% Ammonium Chloride) Birex SE Phenolic with detergent (7.7% O-Phenylphenol, 7.6% p-tertiary amylphenol) 	<ul style="list-style-type: none"> Discide (63.25% Isopropyl Alcohol) Volo (41.6% Isopropyl Alcohol) Lysol IC (0.1% Alkyl Dimethyl Benzyl Ammonium Saccharinate, 79% Ethanol) 								
Limitations on Processing	<ul style="list-style-type: none"> Repeated processing has minimal effect; end of life is normally determined by wear and damage due to use. See Maintenance, Inspection and Testing. Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. 									
Point of Use	<ul style="list-style-type: none"> Ultrasonic inserts must be removed from the handpiece before processing. Do not allow residue or any form of contamination to dry on device. Remove excess soil by wiping device with a disposable wipe under running tepid (potable) water, checking for signs of visible contamination. Dry thoroughly with a disposable wipe. 									
Preparation for decontamination	<ul style="list-style-type: none"> Protect from damage. Following use, device should be reprocessed as soon as is reasonably practical. Device left standing wet may stain or corrode. 									
Automated Cleaning & Thermal Disinfection	<ul style="list-style-type: none"> Use only a properly maintained, inspected, calibrated and approved washer/disinfecter according to ISO 15883 displaying an effective validation (e.g. FDA certification or CE mark, accordingly). Perform a cleaning program cycle in an automated instrument washer, with the recommended program cycle: <table border="1" style="width: 100%; border-collapse: collapse;"> <tr> <td style="width: 33%;">• 2 minutes pre-wash/rinse in cold tap water</td> <td style="width: 33%;">• 2 minutes enzymatic neutralization in hot water at 60°C</td> <td style="width: 33%;">• 2 minutes rinse in hot tap water</td> </tr> <tr> <td>• 5 minutes enzymatic wash in warm water > 43°C</td> <td></td> <td>• 10 minutes drying at 90°C</td> </tr> </table> It is recommended to use sterile or low-germ (< 10 CFU/ml) and endotoxin-free water (< 0.25 EU/ml, e.g. high purity water HPW) for the final rinse phase. Follow cleaning detergent manufacturer's instructions, observing concentration rates and contact times. Loading: Do not exceed 279°F/137°C when drying as part of the washer / disinfecter cycle. Unloading: Inspect for complete removal of visible soil, including holes / cannulas. If soil is still visible after processing, repeat cycle. 	• 2 minutes pre-wash/rinse in cold tap water	• 2 minutes enzymatic neutralization in hot water at 60°C	• 2 minutes rinse in hot tap water	• 5 minutes enzymatic wash in warm water > 43°C		• 10 minutes drying at 90°C			
• 2 minutes pre-wash/rinse in cold tap water	• 2 minutes enzymatic neutralization in hot water at 60°C	• 2 minutes rinse in hot tap water								
• 5 minutes enzymatic wash in warm water > 43°C		• 10 minutes drying at 90°C								
Manual Cleaning	<p>Cleaning</p> <ul style="list-style-type: none"> Follow manufacturer instructions for cleaning detergent, observing concentration rates and contact times. Remove soil with cleaning detergent while wiping with a disposable wipe until free of visible soil. <p>Rinsing</p> <ul style="list-style-type: none"> Remove detergent from device by placing device under running tepid (potable) water and wiping with a disposable wipe for 30 seconds. Dry device with a disposable wipe. 									
Manual Disinfection	<p>Disinfecting</p> <ul style="list-style-type: none"> After cleaning thoroughly, wipe all device surfaces with a disposable wipe in combination with a bactericidal, virucidal, and fungicidal instrument disinfection solution (approved according to local regulations and used according to device disinfectant solution manufacturer's Instructions for Use), observing concentration rates and contact times. <p>Rinsing</p> <ul style="list-style-type: none"> Remove disinfectant solution residue by placing the device under running tepid (potable) water, careful to follow disinfectant manufacturer's instructions. Dry device with a disposable wipe. 									
Manual Drying	<ul style="list-style-type: none"> Dry thoroughly with a disposable wipe. 									
Maintenance, Inspection and Testing	<ul style="list-style-type: none"> Inspect to ensure all visible contamination has been removed. Check for damage and wear: corrosion, pitting, discoloration, cracking, edges should be free of nicks. Discard damaged or corroded product. If not visibly clean, repeat the process or safely dispose of device. 									
Packaging	<ul style="list-style-type: none"> Place product in a paper or paper/plastic steam sterilization pouch. Ensure pack is large enough to contain the product without stressing the seals. Product may be loaded into dedicated instrument trays or general-purpose sterilization trays. 									
Sterilization	<ul style="list-style-type: none"> Place bagged instruments into the steam sterilizer per manufacturer's instructions. <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th colspan="3">*Steam Sterilization Cycles</th> </tr> <tr> <th style="width: 40%;">ITEMS</th> <th style="width: 30%;">STERILIZING TEMPERATURE</th> <th style="width: 30%;">STERILIZING CYCLE TIME</th> </tr> </thead> <tbody> <tr> <td>Steri-Mate 360 Handpiece</td> <td>270°F / 132°C</td> <td>10 minutes</td> </tr> </tbody> </table> <p>*The steam sterilization parameters specified above are for Dynamic Air Removal steam sterilizers (pre-vacuum and steam flush pressure pulse).</p> <p>Alternate Method:</p> <ul style="list-style-type: none"> Place non-bagged instruments into the steam sterilizer and run at the listed cycles. Note: Instruments sterilized unbagged should be used immediately. <p>Drying:</p> <ul style="list-style-type: none"> To dry, use the drying cycle of the sterilizer. Set cycle for 20-30 minutes. 	*Steam Sterilization Cycles			ITEMS	STERILIZING TEMPERATURE	STERILIZING CYCLE TIME	Steri-Mate 360 Handpiece	270°F / 132°C	10 minutes
*Steam Sterilization Cycles										
ITEMS	STERILIZING TEMPERATURE	STERILIZING CYCLE TIME								
Steri-Mate 360 Handpiece	270°F / 132°C	10 minutes								
Storage	<ul style="list-style-type: none"> To maintain sterility, instruments should remain bagged until ready for use and stored in a dry, clean location. 									
Additional Information	<ul style="list-style-type: none"> When sterilizing multiple instruments in one autoclave cycle, ensure that the sterilizer's maximum load is not exceeded. The instructions provided have been validated by the manufacturer of the medical device as being capable of preparing a medical device for re-use. It remains the responsibility of the processor to ensure that the reprocessing is actually performed using equipment, materials and personnel in the reprocessing facility to fully achieve the desired result. This normally requires validation and routine monitoring of the process. 									

Steri-Mate & Jet-Mate Handpieces, Handpiece Adapters and Reservoir Bottles Infection Control Procedures										
Warnings	<ul style="list-style-type: none"> Instruments are not sterile upon receipt and must be sterilized prior to use in accordance with the following instructions. The colorant used in the Steri-Mate Handpiece Soft Grips may bleed during autoclaving. This will not affect performance. Do not disinfect or autoclave the Steri-Mate Handpiece or Steri-Mate Handpiece Adapter while assembled. 									
Limitations on Processing	<ul style="list-style-type: none"> Repeated cleaning and sterilization cycles have minimum effect on these instruments. End of life in normally determined by wear and damage due to use. Reservoir bottle storage caps are not steam autoclave sterilizable. Do not use chemical disinfectants prior to sterilization or rapid deterioration of the material may occur. Automated washer-disinfectors, Cold liquid disinfection/sterilization, chemical vapor sterilization, and dry heat sterilization methods have not been tested or validated for efficacy and are not recommended for use. Do not immerse handpieces or handpiece adapter in ultrasonic bath. 									
Point of Use	<ul style="list-style-type: none"> Before processing, remove Ultrasonic insert from the handpiece and disconnect handpiece from the scaler handpiece cable. Remove any moist debris/residue on the handpiece using a lint free wipe (disposable cloth or paper towel) moistened with water. Then dry handpiece with a fresh dry paper towel or disposable cloth. Alternatively, use a disposable disinfectant cleaning wipe (such as Volowipes™ Disinfecting Cleaning Wipes) to remove moist debris/residue. Do not allow residue or any form of contamination to dry on device. 									
Containment and Transportation	<ul style="list-style-type: none"> Protect instruments from damage. Instruments should be reprocessed as soon as is reasonably practical following use. Instruments left standing wet may stain or corrode. 									
Manual Cleaning	<ul style="list-style-type: none"> Rinse all instruments under running water to remove any gross debris. Wipe sterilizable handpieces, reservoir bottles and storage caps, blue interface caps and sip tubes with a cloth saturated with multi-purpose cleaner and rinse with running water. Discard used cloth. Dry with a clean cloth. Follow manufacturer instructions for cleaning detergent observing concentration rates and contact times. To remove contamination thoroughly wipe the handpiece using soap and water or a cleaning solution (such as Resurge® Instrument Cleaning Solution); pay special attention to holes/cannulas - these can be cleaned using a disposable syringe filled with soap and water or cleaning solution (such as Resurge® Instrument Cleaning Solution). 									
Preparation for Decontamination	<ul style="list-style-type: none"> Remove excess powder from the air polishing or air polishing/scaling handpieces by inserting the applicable cleaning wire(s) provided with your system or handpiece into the powder ports and moving the wire back and forth several times. Remove wire for cleaning and sterilization. 									
Automated Cleaning	<ul style="list-style-type: none"> Reservoir bottles can be washed in a dishwasher. 									
Rinsing	<ul style="list-style-type: none"> Rinse the handpiece under tepid (potable) running water for 30 seconds to remove cleaning agent; a disposable syringe may be used to rinse the holes/cannulas. 									
Drying	<ul style="list-style-type: none"> Dry the handpiece using a dry, lint-free wipe (disposable cloth or paper towel). Check that all debris has been removed. If not, repeat the cleaning process. 									
Disinfection	<ul style="list-style-type: none"> To disinfect the storage caps, generously spray disinfectant solution on a clean cloth and wipe the caps thoroughly. Discard used cloth. Allow disinfectant to air dry. Storage caps are not steam autoclave sterilizable. Disinfection of the sterilizable handpieces, handpiece adapters, reservoir bottles, blue interface caps and sip tubes is not necessary prior to steam autoclave sterilization. 									
Packaging	<ul style="list-style-type: none"> Place instrument in a paper or paper/plastic steam sterilization pouch. Instruments may be loaded into dedicated instrument trays. 									
Sterilization	<ul style="list-style-type: none"> Place bagged instruments into the steam sterilizer per manufacturer's instructions. <table border="1" style="width: 100%; border-collapse: collapse;"> <thead> <tr> <th colspan="3">*Steam Sterilization Cycles</th> </tr> <tr> <th style="width: 40%;">ITEMS</th> <th style="width: 30%;">STERILIZING TEMPERATURE</th> <th style="width: 30%;">STERILIZING CYCLE TIME</th> </tr> </thead> <tbody> <tr> <td>Jet-Mate Handpiece Black Steri-Mate Handpiece Handpiece Adapters Reservoir Bottles Blue Interface Caps with Sip Tubes Gray Steri-Mate Handpiece</td> <td>270°F / 132°C</td> <td>3 minutes</td> </tr> </tbody> </table> <p>*The steam sterilization parameters specified above are for Dynamic Air Removal steam sterilizers (pre-vacuum and steam flush pressure pulse).</p> <p>Alternate method:</p> <ul style="list-style-type: none"> Place non-bagged instruments into the steam sterilizer and run at the listed cycles. Note: Instruments sterilized unbagged should be used immediately. <p>Drying:</p> <ul style="list-style-type: none"> To dry, use the drying cycle of the sterilizer. Set cycle for 20-30 minutes. 	*Steam Sterilization Cycles			ITEMS	STERILIZING TEMPERATURE	STERILIZING CYCLE TIME	Jet-Mate Handpiece Black Steri-Mate Handpiece Handpiece Adapters Reservoir Bottles Blue Interface Caps with Sip Tubes Gray Steri-Mate Handpiece	270°F / 132°C	3 minutes
*Steam Sterilization Cycles										
ITEMS	STERILIZING TEMPERATURE	STERILIZING CYCLE TIME								
Jet-Mate Handpiece Black Steri-Mate Handpiece Handpiece Adapters Reservoir Bottles Blue Interface Caps with Sip Tubes Gray Steri-Mate Handpiece	270°F / 132°C	3 minutes								
Maintenance, Inspection and Testing	<ul style="list-style-type: none"> Visually inspect to ensure all contamination is removed. Check for distortion, damage and wear. Discard damaged, worn or corroded instruments. Inspect o-rings on the handpiece and handpiece cable for damage or wear. Replace if necessary. Ensure all o-rings are in place. 									
Storage	<ul style="list-style-type: none"> To maintain sterility, instruments should remain bagged until ready for use and stored in a dry, clean environment. 									
Manufacturer Contact	<ul style="list-style-type: none"> In the United States, contact Dentsply Sirona Customer Service Technical Support at 800-989-8826. For areas outside the United States, contact your local Dentsply Sirona Representative. 									

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