

MRI-Ready Systems Manual

MRI Procedure Information for the St. Jude Medical™ MR Conditional System

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.



WARNING: This product can expose you to chemicals including ethylene oxide, which is known to the State of California to cause cancer and birth defects or other reproductive harm. For more information, go to www.P65Warnings.ca.gov.

Unless otherwise noted, TM indicates that the name is a trademark of, or licensed to, St. Jude Medical or one of its subsidiaries. ST. JUDE MEDICAL and the nine-squares symbol are trademarks and service marks of St. Jude Medical, LLC and its related companies.

Pat. <http://patents.sjm.com>

© 2018 St. Jude Medical, LLC. All Rights Reserved.

Contents

Introduction	1
Symbols.....	1
St. Jude Medical™ MRI Safety Information.....	1
MRI Scan Parameters for St. Jude Medical MR Conditional Systems	2
St. Jude Medical MR Conditional Systems Device/Lead Combinations.....	3
Instructions for Cardiac Physicians and Clinicians.....	5
I. Confirm that the Patient has an MR Conditional System	5
II. Confirm that No Adverse Conditions to MRI Scanning are Present.....	5
III. Review the Potential Adverse Events.....	5
IV. Generate a Report of the Patient's Permanently Programmed Parameters	6
V. Select and Save MRI Settings	6
VI. Review the MRI Checklist	9
VII. Disable the MRI Settings	11
Instructions for Radiologists and MRI Technologists.....	12
I. Confirm that the Patient has an MR Conditional System	12
II. Confirm that No Adverse Conditions to MRI Scanning are Present.....	12
III. Review the Potential Adverse Events.....	12
IV. Select the Correct Scan Parameters.....	13
V. Check the MRI Settings Status.....	13
VI. Perform the Scan and Monitor the Patient	15
VII. Disable the MRI Settings	15
Technical Support.....	15

Introduction

This manual explains the procedures and precautions that must be followed when scanning a patient who is implanted with a St. Jude Medical™ MR Conditional system.

It is important to read the information in this manual before conducting an MRI scan on a patient with an implanted St. Jude Medical MR Conditional system. Contact Technical Support if you have any questions (page 15).

Refer to the Merlin™ Patient Care System (PCS) on-screen help or to the appropriate device or lead user's manual for non-MRI related information.

The St. Jude Medical MR Conditional system includes a St. Jude Medical MR Conditional device connected to one or more St. Jude Medical MR Conditional leads. For a list of the device/lead combinations that have been tested, refer to the St. Jude Medical MR Conditional Systems device/lead combination tables (page 3).


Testing has demonstrated that the St. Jude Medical MR Conditional system is conditionally safe for use in the MRI environment when used according to the instructions in this manual.

A patient with this system may be safely scanned under the conditions given in this manual. Scanning under other conditions may result in severe patient injury, death, or device malfunction.

Enable the MRI Settings to turn on a mode of operation that allows a patient with an MR Conditional system to be safely scanned by an MRI scanner when used according to the instructions in this manual.

Symbols

Table 1. MR Conditional symbols

Symbol	Description
	Device has been demonstrated to pose no known hazards in a specified MRI environment with specified conditions of use.

St. Jude Medical™ MRI Safety Information

Personnel planning or performing an MRI scan of a patient with an MR Conditional system must follow the procedures and restrictions outlined in this manual.

Failure to follow the procedures outlined here may result in serious harm to the device and the patient.

MRI Scan Parameters for St. Jude Medical MR Conditional Systems

When performing an MRI scan on a patient with a St. Jude Medical™ MR Conditional system, the following scan parameters must be followed.

Table 2. MRI scan parameters

Scan parameters	Settings
Scanner type	Cylindrical-bore magnet, horizontal field orientation
Magnet strength	1.5 Tesla/64 MHz excitation frequency (hydrogen atom only)
Spatial field gradient	≤30 T/m (3000 G/cm)
MR operating mode	Normal Operating Mode
Whole body SAR (Specific Absorption Rate)	≤2 W/kg
Head SAR	≤3.2 W/kg
Gradient slew rate	≤200 T/m/s per axis
Scan region	Full-body

The following RF coil types may be used when performing an MRI scan:

1. Full-body RF transmit coil with any receive coil in Normal Operating Mode
2. Local RF transmit-receive coil in Normal Operating Mode:
 - RF transmit-receive Head coil (quadrature only)
 - RF transmit-receive Lower extremity coil (quadrature only)
 - RF transmit-receive Upper extremity coil (quadrature only)

WARNING: Use of local RF transmit coils placed directly over the device is untested.

CAUTION: Multiple leads can be connected to an MR Conditional device. Not all lead lengths are MR Conditional. Each lead needs to be checked for MRI compatibility and individual scan parameters. The device/lead combination tables list the MR Conditional lead lengths (page 3).

St. Jude Medical MR Conditional Systems Device/Lead Combinations

Pacemakers

Table 3. St. Jude Medical MR Conditional System device/lead combinations for Accent MRI™ pacemakers

Device model	Lead model (lengths)	RF Power (SAR)	Scan region
Accent MRI PM1224 PM2218	Tendril MRI™ LPA1200M (46 cm, 52 cm, 58 cm)	Normal Operating Mode	Full-body

Table 4. St. Jude Medical MR Conditional System device/lead combinations for Assurity MRI™ and Endurity MRI™ pacemakers

Device model	Lead model (lengths)	RF Power (SAR)	Scan region
Assurity MRI PM1272 PM2272	Tendril MRI™ LPA1200M (46 cm, 52 cm, 58 cm)	Normal Operating Mode	Full-body
Endurity MRI PM1172 PM2172	Tendril™ STS 2088TC (46cm, 52cm, 58cm)		

CRT-Ps

Table 5. St. Jude Medical MR Conditional System device/lead combinations for Quadra Allure™ and Quadra Allure MP™ CRT-Ps

Device model	Lead model (lengths)	RF Power (SAR)	Scan region
Quadra Allure PM3542	Quartet™ 1456Q (86 cm)	Normal Operating Mode	Full-body
Quadra Allure MP PM3562	1457Q (86 cm)		
	1458Q (86 cm) 1458QL (86 cm)		
	Tendril™ STS 2088TC (46 cm, 52 cm, 58 cm)		

ICDs

Table 6. St. Jude Medical MR Conditional System device/lead combinations for Ellipse™ ICDs

Device model	Lead model (lengths)	RF Power (SAR)	Scan region
Ellipse CD1311-36Q CD1411-36Q CD1411-36QC CD2311-36Q CD2411-36Q CD2411-36QC	Durata™ 7120Q (58 cm, 65 cm) 7122Q (58 cm, 65 cm)	Normal Operating Mode	Full-body
	Optisure™ LDA220Q (58 cm, 65 cm) LDA210Q (58 cm, 65 cm)		
	Tendril MRI™ LPA1200M (46 cm, 52 cm)		
	Tendril™ STS 2088TC (46cm, 52cm)		

Table 7. St. Jude Medical MR Conditional System device/lead combinations for Fortify Assura™ ICDs

Device model	Lead model (lengths)	RF Power (SAR)	Scan region
Fortify Assura CD1357-40Q CD1357-40QC CD2357-40Q CD2357-40QC	Durata™ 7120Q (58 cm, 65 cm) 7122Q (58 cm, 65 cm) <hr/> Optisure™ LDA220Q (58 cm, 65 cm) LDA210Q (58 cm, 65 cm) <hr/> Tendril MRI™ LPA1200M (46 cm, 52 cm) <hr/> Tendril™ STS 2088TC (46cm, 52cm)	Normal Operating Mode	Full-body

CRT-Ds

Table 8. St. Jude Medical MR Conditional System device/lead combinations for Quadra Assura™ and Quadra Assura MP™ CRT-Ds

Device model	Lead model (lengths)	RF Power (SAR)	Scan region
Quadra Assura CD3365-40Q CD3365-40QC Quadra Assura MP CD3369-40Q CD3369-40QC	Quartet™ 1456Q (86 cm) 1457Q (86 cm) 1458Q (86 cm) 1458QL (86 cm) <hr/> Durata™ 7120Q (58 cm, 65 cm) 7122Q (58 cm, 65 cm) <hr/> Optisure™ LDA210Q (58 cm, 65 cm) LDA220Q (58 cm, 65 cm) <hr/> Tendril MRI™ LPA1200M (46 cm, 52 cm) <hr/> Tendril™ STS 2088TC (46cm, 52cm)	Normal Operating Mode	Full-body

Instructions for Cardiac Physicians and Clinicians

NOTE: Radiologists and MRI technologists should see Instructions for Radiologists and MRI Technologists (page 12).

The role of cardiac physicians and clinicians in preparing a patient for an MRI scan is to:

- Confirm that the Patient has an MR Conditional System (page 5)
- Confirm that No Adverse Conditions to MRI Scanning are Present (page 5)
- Review the Potential Adverse Events (page 5)
- Generate a Report of the Patient's Permanently Programmed Parameters (page 6)
- Select and Save MRI Settings (page 6)
- Review the MRI Checklist (page 9)
- Disable the MRI Settings (page 11)

I. Confirm that the Patient has an MR Conditional System

1. Review the patient's ID card or Parameter report (generated by the Merlin™ PCS) to obtain the model numbers for both the implanted lead(s) and device.
2. Check the model numbers against the St. Jude Medical™ MR Conditional Systems Device/Lead Combination tables (page 3).

NOTE: Multiple leads can be connected to an MR Conditional device. Not all lead lengths are MR Conditional. Each lead needs to be checked for MRI compatibility and individual scan parameters.


II. Confirm that No Adverse Conditions to MRI Scanning are Present

If any conditions exist that could make MRI scanning unsafe, do not scan the patient. Such conditions include:

- Patient has elevated body temperature or compromised thermoregulation at time of scan
- The device is at End-of-Life
- A combination of one or more leads and a device that is not listed as MR Conditional in the St. Jude Medical™ MR Conditional Systems Device/Lead Combination tables (page 3)
- Broken or intermittently functioning St. Jude Medical MR Conditional leads
- Lead impedance measurements not within the programmed lead impedance limits
- Additional hardware including lead extenders, lead adapters, or abandoned leads
- A St. Jude Medical MR Conditional system implanted in sites other than the left and right pectoral region (see figure below for correct locations of device implant)
- Patients with unstable capture thresholds
- Patients with capture threshold values of >2.5 V at a pulse width of 0.5 ms for RA and RV leads
- Patients with capture threshold values of >2.0 V at a pulse width of 0.5 ms for the LV lead
- Complaints of diaphragmatic stimulation at a pacing output of 5.0 V or 7.5 V and at a pulse width of 1.0 ms in patients whose device will be programmed to an asynchronous pacing mode when MRI Settings are enabled
- For ICDs and CRT-Ds: Capacitor has not been prepared for the MRI scan (see Review the MRI Checklist)

NOTE: Lead fractures or other damage to the leads may cause changes in the electrical properties of the St. Jude Medical MR Conditional system that make the system unsafe for an MRI scan. Patients with damaged leads may be harmed if an MRI scan is performed.

Figure 1. Correct locations for device implant

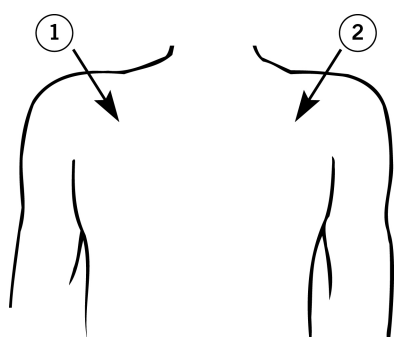

1. Right-pectoral region
2. Left-pectoral region

III. Review the Potential Adverse Events

The St. Jude Medical™ MR Conditional system has been designed to minimize the potential adverse events that may cause patient harm. The following potential adverse events may occur in the MRI environment:

- Lead electrode heating and tissue damage resulting in loss of sensing or capture or both
- Device heating resulting in tissue damage in the implant pocket or patient discomfort or both
- Induced currents on leads resulting in continuous capture, VT/VF, hemodynamic collapse, or all three
- Damage to the device or leads causing the system to fail to detect or treat irregular heartbeats or causing the system to treat the patient's condition incorrectly
- Damage to the functionality or mechanical integrity of the device resulting in the inability to communicate with the device
- Movement or vibration of the device or leads
- Lead dislodgement
- Competitive pacing and potential for VT/VF induction if asynchronous pacing is programmed when MRI Settings are enabled
- Syncope due to loss of pacing if no pacing support is programmed with MRI Settings
- For ICDs and CRT-Ds: Death due to untreated spontaneous arrhythmia because tachy therapy is disabled when MRI Settings are programmed

Potential interactions between the MRI scanner and the MR Conditional system include:

- The magnetic material of an implanted system may exert force, vibration, and torque effects due to the static magnetic field and gradient magnetic fields produced by an MRI scanner. These effects have been shown to be minimal in St. Jude Medical MR Conditional systems. Patients may feel a mild tugging or vibration sensation at the site of the device implant while in or near the MRI scanner.
- The gradient magnetic and RF fields produced by an MRI scanner could potentially interact with the MR Conditional system and cause unintended stimulation of the heart. When all conditions outlined in this manual are met, the voltages and pulse widths induced on the leads of the St. Jude Medical MR Conditional System are limited so that the potential for capturing the heart is minimized.
- The RF fields generated by an MRI scanner could induce voltages onto an implanted lead system that may cause heating at the lead electrodes. This heating could damage the tissue surrounding the electrodes and compromise pacing and sensing thresholds at that site. When all conditions outlined in this manual are met, St. Jude Medical MR Conditional leads have been tested and shown to limit heating at the electrodes and to minimize thermal damage of the surrounding cardiac tissue.

IV. Generate a Report of the Patient's Permanently Programmed Parameters

CAUTION: Do not bring the Merlin™ Patient Care System (PCS) Model 3650 or the SJM MRI Activator™ handheld device Model EX4000 into Zone IV (MRI scanner magnet room), as defined by the American College of Radiology. Each device is MR Unsafe.

1. Interrogate the device with the Merlin™ PCS.
2. If needed, perform capture, sense, and lead impedance tests.
3. From the FastPath™ Summary screen, select the Print button to print the Diagnostics and any other relevant reports. The Merlin PCS will print to the default printer (internal printer, external printer or PDF).

NOTE: Device diagnostic data may be suspended or cleared when MRI Settings are enabled. Refer to the table below for the behavior of each system.

Table 9. Diagnostic data during MRI Settings

Device family	Diagnostics
Pacemakers	
Accent MRI™ pacemakers	Suspended with MRI Settings enabled
Assurity MRI™ pacemakers	Suspended with MRI Settings enabled
Endurity MRI™ pacemakers	Suspended with MRI Settings enabled
CRT-Ps	
Quadra Allure™ CRT-Ps	Suspended with MRI Settings enabled
Quadra Allure MP™ CRT-Ps	Suspended with MRI Settings enabled
ICDs	
Ellipse™ ICDs	Cleared from memory when MRI Settings are programmed
Fortify Assura™ ICDs	Cleared from memory when MRI Settings are programmed
CRT-Ds	
Quadra Assura™ CRT-Ds	Cleared from memory when MRI Settings are programmed
Quadra Assura MP™ CRT-Ds	Cleared from memory when MRI Settings are programmed

For any device, it is recommended that the clinician perform a complete follow-up prior to the MRI procedure to save all diagnostic data.

V. Select and Save MRI Settings

NOTE: The Merlin™ PCS must be operating with software version 24.3.1 or greater to interrogate an MR Conditional device.

The MRI parameter settings are selected at the physician's discretion.

The default MRI parameter settings are automatically stored in the St. Jude Medical™ MR Conditional device.

Table 10. Default MRI Settings for Pacing

Parameter	Setting
MRI mode (dual-chamber pacemakers)	D00
MRI mode (single-chamber pacemakers)	V00 or A00 (as applicable)
MRI base rate	85 bpm
MRI paced AV delay	120 ms
MRI pulse amplitude	5.0 V
MRI pulse width	1.0 ms
MRI pulse configuration	Bipolar

Table 11. Default MRI Settings for CRT-Ps

Parameter	Setting
MRI mode	D00
MRI base rate	85 bpm
MRI paced AV delay	120 ms
MRI pulse amplitude	5.0 V
MRI pulse width	1.0 ms
MRI pulse configuration	Bipolar
MRI V pacing chamber	RV only

Table 12. Default MRI Settings for ICDs

Parameter	Setting
MRI mode	Pacing Off
MRI base rate	n/a
MRI paced AV delay	n/a
MRI pulse amplitude	n/a
MRI pulse width	n/a
MRI pulse configuration	n/a
Tachy therapy	Disabled

Table 13. Default MRI Settings for CRT-Ds

Parameter	Setting
MRI mode	D00
MRI base rate	85 bpm
MRI paced AV delay	120 ms
MRI pulse amplitude	5.0 V
MRI pulse width	1.0 ms
MRI pulse configuration	Bipolar
MRI V pacing chamber	RV Only
Tachy therapy	Disabled

If you change the MRI Settings from the default values, you must save the modified MRI Settings in the device as described below. Refer to the Merlin PCS on-screen help for more information on selecting, testing, and saving the MRI parameter settings.

1. After you interrogate the device with the Merlin PCS, select the Parameters button on the right to open the Parameters window. Then, select the MRI Settings tab. This opens the MRI Settings window.
2. From this window, you can modify the default values of the MRI parameters that are in effect when the MRI Settings are enabled.
3. For ICDs and CRT-Ds, select the appropriate HV lead type implanted in the patient.
 - For patients implanted with dual coil defibrillation lead, select Dual Coil value.

- For patients implanted with single coil defibrillation lead, select the Single Coil value.

NOTE: Selecting the correct HV lead type ensures appropriate testing during MRI setup.

4. You can temporarily test the settings if you select the Test MRI Settings button. Use this function to evaluate the patient's hemodynamic status with the proposed MRI parameter settings.
5. Select the Cancel Test button to return to permanently programmed settings.
6. Select the Save MRI Settings button to save any changed parameters.

NOTE: MRI Pacing Chamber defaults to RV only in CRT-P and CRT-D devices. Left Ventricular pacing will be turned off when MRI Settings are programmed.

7. If you plan to use the SJM MRI Activator™ handheld device to implement the MRI Settings, you must review the MRI Activator Checklist and enable the pulse generator to communicate with the SJM MRI Activator handheld device. See Enabling the Device to Communicate with the SJM MRI Activator handheld device (page 8).
8. When you are satisfied with the MRI Settings, select the Setup for MRI Now button to open the MRI Checklist.

CAUTION: Regardless of the programmed permanent pacing mode, sensed events are ignored by the device when MRI Settings are enabled. Determine whether or not pacing support is needed during the MRI scan. When pacing support is needed, set the MRI Mode to an asynchronous pacing mode (DOO, AOO, or VOO). When pacing support is not needed, set the MRI Mode to Pacing Off.

Some patients may be susceptible to cardiac arrhythmia induced by competitive pacing when an asynchronous MRI Mode is selected. For these patients, it is important to select an appropriate MRI pacing rate to avoid competitive pacing and then minimize the duration of the asynchronous pacing operation.

In MR Conditional ICDs and CRT-Ds, tachycardia therapy is disabled when the MRI Settings are programmed.

Enabling the Device to Communicate with the SJM MRI Activator™ handheld device

To use the SJM MRI Activator™ handheld device, you must first enable the pulse generator to communicate with the SJM MRI Activator handheld device. This is done through the MRI Settings window on the Merlin™ PCS.

NOTE: Before communication between the SJM MRI Activator handheld device and the pulse generator can be enabled, you must obtain in-range bipolar pacing lead impedance measurements from the current programming session. If you plan to use the SJM MRI Activator handheld device to implement MRI Settings, program V. AutoCapture "on" prior to enabling MRI Settings. The AutoCapture feature will provide a safety margin against increased pacing capture thresholds that may occur post-MRI due to lead tip heating.

To enable the pulse generator to communicate with the SJM MRI Activator handheld device:

1. Determine the appropriate MRI Settings as described above.
2. From the MRI Settings window, select the Setup MRI Activator button.
The MRI Activator Checklist window opens.
3. Review each condition on the checklist and check off each one that applies. You will not be able to enable the pulse generator to communicate with the SJM MRI Activator handheld device until all boxes are checked.
Once you have verified each condition, the Enable MRI Activator button becomes available.
4. Select the Enable MRI Activator button.
The device can now communicate with the SJM MRI Activator handheld device.

Table 14. St. Jude Medical™ MR Conditional pulse generators with SJM MRI Activator™ handheld device functionality

Model name and number	Operates with the SJM MRI Activator handheld device Model EX4000
Pacemakers	
Accent MRI™ pacemakers PM1224, PM2218	Yes
Assurity MRI™ pacemakers PM1272, PM2272	Yes
Endurity MRI™ pacemakers PM1172, PM2172	Yes
CRT-Ps	
Quadra Allure™ CRT-Ps PM3542	No
Quadra Allure MP™ CRT-Ps PM3562	No
ICDs	

Table 14. St. Jude Medical™ MR Conditional pulse generators with SJM MRI Activator™ handheld device functionality

Model name and number	Operates with the SJM MRI Activator handheld device Model EX4000
Ellipse™ ICDs CD1311-36Q, CD1411-36Q, CD1411-36QC, CD2311-36Q, CD2411-36Q, CD2411-36QC	No
Fortify Assura™ ICDs CD1357-40Q, CD1357-40QC, CD2357-40Q, CD2357-40QC	No
CRT-Ds	
Quadra Assura™ CRT-Ds CD3365-40Q, CD3365-40QC	No
Quadra Assura MP™ CRT-Ds CD3369-40Q, CD3369-40QC	No

VI. Review the MRI Checklist

Figure 2. An example of the MRI Checklist screen for pacemakers on the Merlin™ PCS

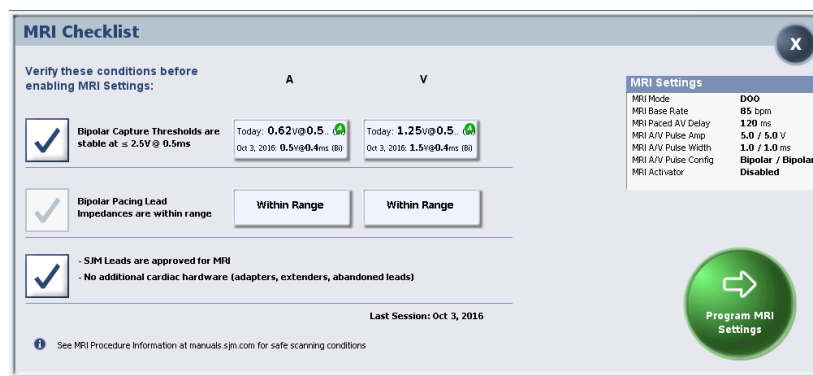


Figure 3. An example of the MRI Checklist screen for CRT-Ps on the Merlin™ PCS

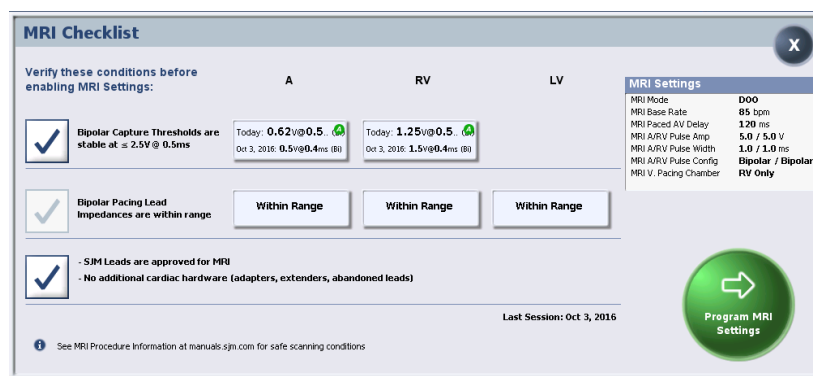


Figure 4. An example of the MRI Checklist screen for ICDs on the Merlin™ PCS


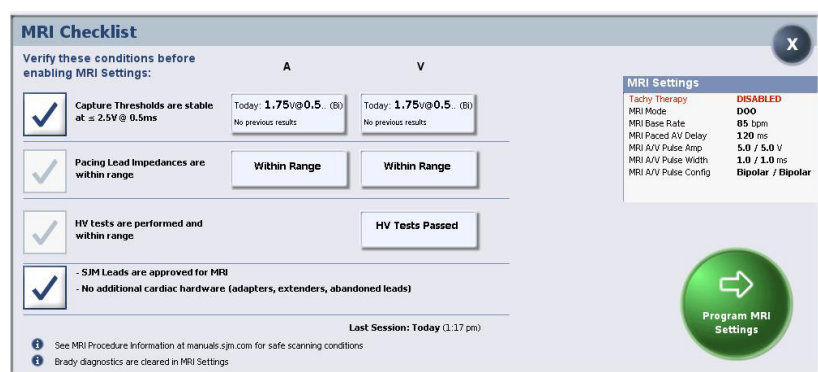


Figure 5. An example of the MRI Checklist screen for CRT-Ds on the Merlin™ PCS

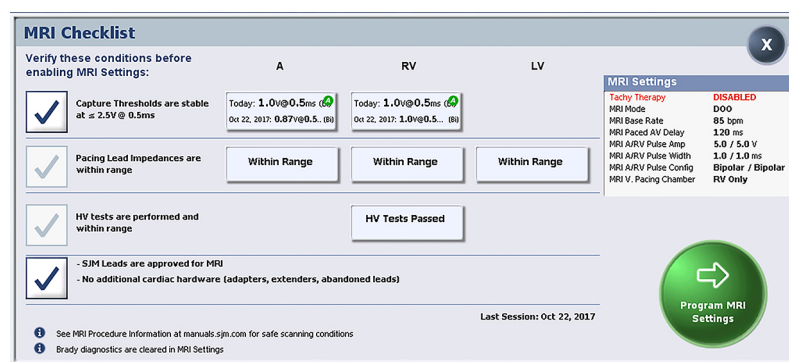

- After you have selected the appropriate MRI Settings, from the MRI Settings window on the Merlin™ PCS, select the Setup for MRI Now button.
After the system performs test measurements required for MRI setup, the MRI Checklist window opens.
- Review each condition on the checklist and check off each one that applies. You will not be able to program MRI Settings until all boxes are checked.
The MRI software provides automatic verification that no device or lead issues are detected that may compromise patient safety during an MRI scan. Before allowing the user to initiate the MRI Settings, the MRI software ensures:
 - Pacing lead impedance is within range - If bipolar pacing lead impedance for RA, RV, or LV leads is out of range, the software prevents the MRI Settings from being enabled.
 - Defibrillation lead impedance is within range - For ICDs and CRT-Ds, if the lead impedance for any of the implanted coils on the defibrillation lead is out of range, the software prevents the MRI Settings from being enabled.
 - Capacitor maintenance is postponed - For ICDs and CRT-Ds, the software prepares the capacitor to prevent automatic capacitor maintenance from occurring during an MRI scan.
- If any of the system integrity tests required for MRI setup is incomplete, Perform Test will be displayed on the test panel in the MRI Checklist. Click on the panel to manually run the incomplete test before enabling MRI Settings.
- Once you have completed the checklist, select the Program MRI Settings button to enable the MRI Settings.
The MRI Settings: Active window appears. This window confirms the programmed changes. Use this window to print the MRI Summary report and end the session (recommended) before performing the MRI scan. MRI Settings can also be disabled in this window.
NOTE: If the programmer Shock button is selected when MRI Settings are enabled, the system will disable the MRI Settings and display the emergency shock dialog box. After an emergency shock, restore the MRI Settings before scanning the patient. Once you have completed the checklist, select the Program MRI Settings button to enable the MRI Settings.
- Select Print MRI Report button to print the report.
- Select End Session.

The patient is now ready for the MRI scan.

CAUTION: An ICD or CRT-D patient must be hemodynamically monitored while MRI Settings are programmed, and an external defibrillator must be kept constantly ready.

Be sure to disable the MRI Settings as soon as the MRI scan is complete.

VII. Disable the MRI Settings

CAUTION: Do not bring the Merlin™ Patient Care System (PCS) Model 3650 or the SJM MRI Activator™ handheld device Model EX4000 into Zone IV (MRI scanner magnet room), as defined by the American College of Radiology. Each device is MR Unsafe.

Immediately following the MRI procedure, the patient's device management physician or clinician must:

1. Interrogate the device using the Merlin PCS or the SJM MRI Activator handheld device.
2. For the SJM MRI Activator handheld device, select the MRI Settings Off button.
3. For the Merlin PCS, disable the MRI Settings by selecting the Disable MRI Settings button. This restores the permanently programmed settings.
 - a. Confirm the permanently programmed settings are appropriate.
 - b. To ensure that an accurate lead impedance measurement, perform a lead impedance test by selecting Tests > Battery & Leads > Update leads.
 - c. Check the pacing capture thresholds after the scan is complete and ensure that the pacing parameters are programmed adequately for the patient based on the threshold.

Refer to the Merlin PCS on-screen help for information on selecting and programming parameter settings.

NOTE: If the device performs an automatic lead impedance measurement during the procedure, the results may be inaccurate. The magnetic field exerted by the MRI scanner can sometimes result in an inaccurate lead impedance measurement.

Instructions for Radiologists and MRI Technologists

NOTE: Cardiac physicians and clinicians should see Instructions for Cardiac Physicians and Clinicians (page 5).

The role of the radiologist or MRI technologist is to:

- Confirm that the Patient has an MR Conditional System (page 12)
- Confirm that No Adverse Conditions to MRI Scanning are Present (page 12)
- Review the Potential Adverse Events (page 12)
- Select the Correct Scan Parameters (page 13)
- Check the MRI Settings Status (page 13)
- Perform the Scan and Monitor the Patient (page 15)
- Disable the MRI Settings (page 15)

I. Confirm that the Patient has an MR Conditional System

1. Review the patient's ID card or the MRI Summary Report (generated by the Merlin™ PCS) to obtain the model numbers for both the implanted lead(s) and device.
2. Check the model numbers against the St. Jude Medical™ MR Conditional Systems Device/Lead Combination tables (page 3).

NOTE: Multiple leads can be connected to an MR Conditional device. Not all lead lengths are MR Conditional. Each lead needs to be checked for MRI compatibility and individual scan parameters.


II. Confirm that No Adverse Conditions to MRI Scanning are Present

If any conditions exist that could make MRI scanning unsafe, do not scan the patient. Such conditions include:

- Patient has elevated body temperature or compromised thermoregulation at time of scan
- A combination of one or more leads and a device that is not listed as MR Conditional in the St. Jude Medical™ MR Conditional Systems Device/Lead Combination tables (page 3)
- Broken or intermittently functioning St. Jude Medical MR Conditional leads
- Additional hardware including lead extenders, lead adapters, or abandoned leads
- A St. Jude Medical MR Conditional system implanted in sites other than the left and right pectoral region (see figure below)
- Any prone or lateral recumbent patient positions, or positions where the patient's arm is raised above his or her head; these are excluded and have not been tested

NOTE: Lead fractures or other damage to the leads may cause changes in the electrical properties of the St. Jude Medical MR Conditional system that make the system unsafe for an MRI scan. Patients with damaged leads may be harmed if an MRI scan is performed.

Figure 6. Correct locations for device implant

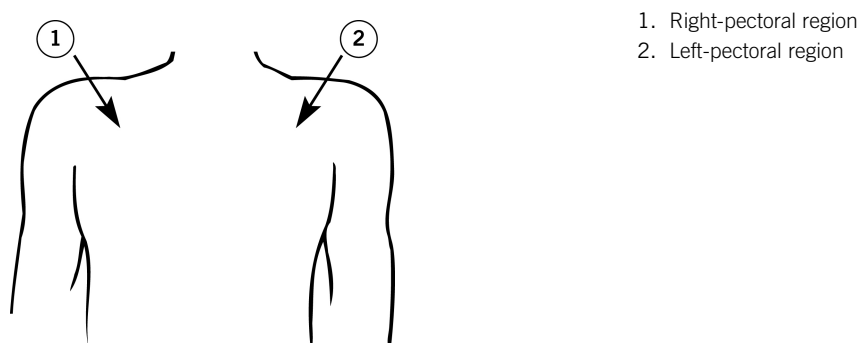

III. Review the Potential Adverse Events

Potential interactions between the MRI scanner and the MR Conditional system include:

- The magnetic material of an implanted system may exert force, vibration, and torque effects due to the static magnetic field and gradient magnetic fields produced by an MRI scanner. These effects have been shown to be minimal in St. Jude Medical MR Conditional systems. Patients may feel a mild tugging or vibration sensation at the site of the device implant while in or near the MRI scanner.
- The gradient magnetic and RF fields produced by an MRI scanner could potentially interact with the MR Conditional system and cause unintended stimulation of the heart. When all conditions outlined in this manual are met, the voltages and pulse widths induced on the leads of the St. Jude Medical MR Conditional System are limited so that the potential for capturing the heart is minimized.

- The RF fields generated by an MRI scanner could induce voltages onto an implanted lead system that may cause heating at the lead electrodes. This heating could damage the tissue surrounding the electrodes and compromise pacing and sensing thresholds at that site. When all conditions outlined in this manual are met, St. Jude Medical MR Conditional leads have been tested and shown to limit heating at the electrodes and to minimize thermal damage of the surrounding cardiac tissue.

IV. Select the Correct Scan Parameters

1. Refer to the MRI scan parameters table for the applicable scan parameter settings for approved MR Conditional device/lead combinations (page 2).
2. Refer to the section on St. Jude Medical™ MR Conditional Systems Device/Lead Combinations (page 3) to identify the settings for RF power (SAR) and scan regions.
3. Make sure that you identify the **combination** of one or more leads and a device to select the correct settings.

V. Check the MRI Settings Status

CAUTION: Do not bring the Merlin™ Patient Care System (PCS) Model 3650 or the SJM MRI Activator™ handheld device Model EX4000 into Zone IV (MRI scanner magnet room), as defined by the American College of Radiology. Each device is MR Unsafe.

If you do not have an SJM MRI Activator handheld device Model EX4000, see Check the MRI Setting Status without the SJM MRI Activator Handheld Device (page 13).

If you do have an SJM MRI Activator handheld device Model EX4000 available, see Check the MRI Settings Status with the SJM MRI Activator Handheld Device (page 13).

Check the MRI Settings Status without the SJM MRI Activator™ Handheld Device

1. Refer to the MRI Summary Report generated by the Merlin™ PCS.
2. Confirm these settings with the device management physician or clinician.

The currently programmed settings should include:

Table 15. MRI Settings¹

Parameter	Setting
Tachy therapy (ICD and CRT-D patients)	Disabled
MRI mode	DOO, VOO, AOO, Pacing Off
MRI base rate	Pacemakers and CRT-Ps: 30–120 bpm ICDs and CRT-Ds: 30–100 bpm
MRI paced AV delay	25–120 ms
MRI pulse amplitude	5.0 or 7.5 V
MRI pulse width	1.0 ms
MRI pulse configuration	Bipolar
MRI V Pacing Chamber (CRT-P and CRT-D patients)	RV only

Check the MRI Settings Status with the SJM MRI Activator™ Handheld Device

To check the status of the MRI Settings:


1. Refer to the table below to determine if the device can communicate with the SJM MRI Activator handheld device.
2. Place the SJM MRI Activator handheld device over the implanted pulse generator. The SJM MRI Activator handheld device should be held against the patient's chest directly over the implanted pulse generator (see figure for correct locations for device implants).
3. Press the MR Status button (see figure of SJM MRI Activator handheld device).



- MRI Settings Enabled. The green LEDs (on the right side of the SJM MRI Activator handheld device) illuminate continuously for 5 seconds.
- MRI Settings Disabled. The red LEDs (on the left side of the SJM MRI Activator handheld device) illuminate continuously for 5 seconds.

¹ This is the entire range of all possible settings for each parameter.

Figure 7. SJM MRI Activator handheld device

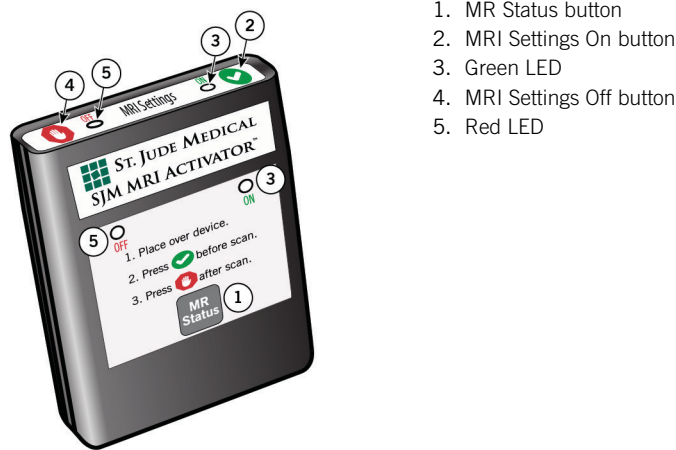

1. MR Status button
2. MRI Settings On button
3. Green LED
4. MRI Settings Off button
5. Red LED

Enable the MRI Settings with the SJM MRI Activator handheld device

If the MRI Settings are not enabled and the patient is ready for the MRI scan, enable the settings with the SJM MRI Activator handheld device.

To enable the MRI Settings:


1. Place the SJM MRI Activator handheld device over the implanted pulse generator. The SJM MRI Activator handheld device should be held against the patient's chest directly over the implanted pulse generator (see figure for correct locations for device implant).
2. Press the MRI Settings On button.



The LEDs may flash before they illuminate continuously.

- When the MRI Settings are enabled, the green LEDs illuminate continuously for 5 seconds to indicate that the device MRI Setting is enabled and the patient can proceed with the MRI scan.
- When the MRI Settings could not be enabled, the red LEDs illuminate continuously for 5 seconds to indicate that an error occurred while enabling MRI Settings in the implanted MR Conditional pacing system, or that the implanted pacing system did not satisfy the conditions for an MRI scan. In this case, the patient should NOT proceed with the MRI scan. Contact the referring physician for further instruction.

Figure 8. Correct locations for device implant

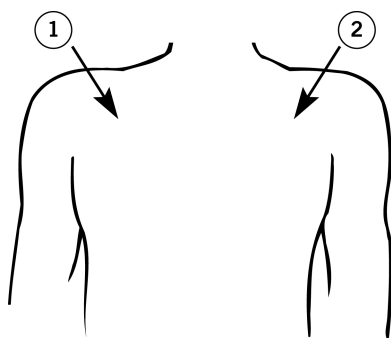

1. Right-pectoral region
2. Left-pectoral region

Table 16. St. Jude Medical™ MR Conditional pulse generators with SJM MRI Activator™ handheld device functionality

Model name and number	Operates with the SJM MRI Activator handheld device Model EX4000
Pacemakers	
Accent MRI™ pacemakers PM1224, PM2218	Yes
Assurity MRI™ pacemakers PM1272, PM2272	Yes

Table 16. St. Jude Medical™ MR Conditional pulse generators with SJM MRI Activator™ handheld device functionality

Model name and number	Operates with the SJM MRI Activator handheld device Model EX4000
Endurity MRI™ pacemakers PM1172, PM2172	Yes
CRT-Ps	
Quadra Allure™ CRT-Ps PM3542	No
Quadra Allure MP™ CRT-Ps PM3562	No
ICDs	
Ellipse™ ICDs CD1311-36Q, CD1411-36Q, CD1411-36QC, CD2311-36Q, CD2411-36Q, CD2411-36QC	No
Fortify Assura™ ICDs CD1357-40Q, CD1357-40QC, CD2357-40Q, CD2357-40QC	No
CRT-Ds	
Quadra Assura™ CRT-Ds CD3365-40Q, CD3365-40QC	No
Quadra Assura MP™ CRT-Ds CD3369-40Q, CD3369-40QC	No

VI. Perform the Scan and Monitor the Patient

Proper patient monitoring must be provided during the MRI scan. This includes continuous monitoring of the patient's hemodynamic function. Since the MR environment may interfere with the patient monitoring system, it is recommended that more than one of the following systems be used: electrocardiography, pulse oximetry, noninvasive blood pressure measurements. If the patient's hemodynamic function is compromised during the MRI scan, discontinue the MRI scan and take the proper measures to restore the patient's hemodynamic function.

Verbal communication with the patient during the MRI scan is recommended.

Keep an external defibrillator available during the MRI scan.

CAUTION: In MR Conditional ICDs and CRT-Ds, tachycardia therapy is disabled when the MRI Settings are programmed.

An ICD or CRT-D patient must be hemodynamically monitored while MRI Settings are programmed, and an external defibrillator must be kept constantly ready.

Be sure to disable the MRI Settings as soon as the MRI scan is complete.

St. Jude Medical™ MR Conditional leads have demonstrated minimal image distortion for areas surrounding the implanted leads when the device is out of the field of view. Significant image distortion can result from the presence of the device within the field of view. Image artifacts and distortion resulting from the presence of the device and the leads within the field of view must be considered when selecting the field of view and imaging parameters. These factors must also be considered when interpreting the MRI images.

VII. Disable the MRI Settings

1. If you are using the SJM MRI Activator™ handheld device, place the SJM MRI Activator handheld device over the implanted device.

The SJM MRI Activator handheld device should be touching the patient's clothing directly over the implanted device.

2. Press the MRI Settings Off button.



The LEDs may flash before they illuminate continuously.

- MRI Settings Disabled. The red LEDs illuminate continuously for 5 seconds.

3. If you are not using the SJM MRI Activator handheld device, the MRI Settings must be disabled by the patient's device management physician or clinician using the Merlin™ PCS.

Technical Support

St. Jude Medical maintains 24-hour phone lines for technical questions and support:

- 1 818 362 6822

- 1 800 722 3774 (toll-free within North America)
- + 46 8 474 4147 (Sweden)
- + 61 2 9936 1200 (Australia)
- manuals.sjm.com

For additional assistance, call your local St. Jude Medical representative.



St. Jude Medical
Cardiac Rhythm
Management Division
15900 Valley View Court
Sylmar, CA 91342 USA
+1 818 362 6822

EC	REP
----	-----

St. Jude Medical
Coordination Center BVBA
The Corporate Village
Da Vincilaan 11 Box F1
1935 Zaventem
Belgium
+32 2 774 68 11

sjm.com



2018-12
ARTEN100166732 A

100166732