

SHEER WHITE WHITENING STRIPS

Professional whitening results in 5 days!



Sheer White whitening strips are 20% carbamide peroxide whitening films that go on easy and stays where you put them. Each Sheer White kit consists of 5 upper and lower professional whitening strips. The thin flexible moldable material closely adapts to every surface holding the whitening gel where it belongs – against the tooth surface. There is no leaking or washing out of gel.

Directions:

1. Remove the cover of plastic tray and make sure the sticky gel side of strips faces up.
2. Position the film so that it is centered on your arch and firmly apply from the center outward. The edge should just extend onto the gums slightly and the excess film can be folded under towards the back of the teeth.
3. Leave on for 1 to 2 hours, for best results they can be worn overnight.
4. When you are ready to remove the strips, rinse lightly with water, then remove the strips from the outer edge towards the middle.
5. After the strips are removed, brush your teeth to remove any excess gel.

Complete directions can be found in each pack of Sheer White

